
Loveline krush

ENGLISH

Enjoy your *Lovelife* krush but first please take a moment to review this manual. ;-)

♥ KRUSH What's included

- ♥ Smart Kegel ball with Bluetooth® App control, vibratory feedback and built-in sensors
- ♥ Positioning anchor
- ♥ USB magnetic charging cable
- ♥ Satin privacy pouch

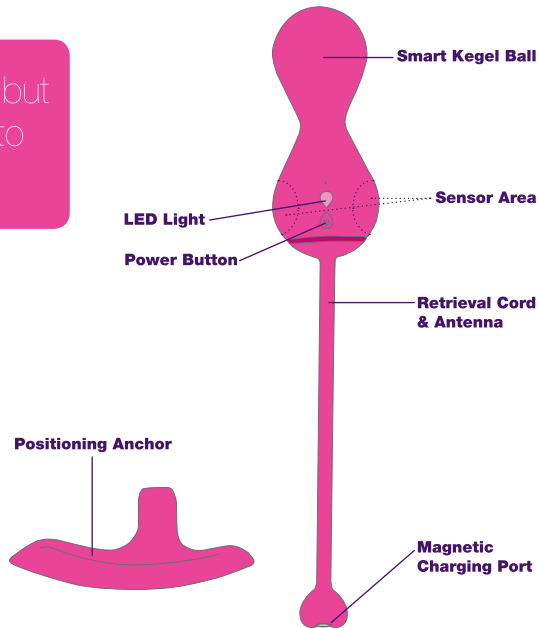♥ Why KRUSH

Krush is a revolutionary way to practice pelvic floor muscle exercises. The larger part of the Kegel ball contains built-in sensors that, when paired with the tasl app, can measure and track pressure, control, endurance and grip of your PC muscle. The tasl app will provide you with real-time visual feedback of your PC muscle activity. This will help you to recognize if you are doing your Kegel exercises correctly. The App will also track your progress and improvement.

Regular Kegel exercise over time can help to tone and strengthen the PC muscle. What is a PC muscle? The "PC" (or Pubococcygeus) muscle stretches from the pubic bone to the tail bone. It helps to keep your bladder, uterus and other internal organs in place. It also assists in the control of opening and closing your urethra, the tube through which urine leaves your body. This muscle also plays a role in core stability. Kegel exercises consist of repeatedly contracting and relaxing the muscles that form part of the pelvic floor. Maintaining a regular exercise routine can aid in the preparation for childbirth and in recovery. It can also improve bladder control keeping incontinence at bay, improve core stability and even improve the intensity of orgasms.

♥ Who should use KRUSH


This product was developed for the personal use of adult women. The user should read through this manual thoroughly prior to use.

Precautions: *Always consult a physician about the safety of exercising with krush or if you are pregnant or have given birth within the past 12 months. Consult a physician if you have any medical concerns prior to using this product. Use only as directed. Do not use if you have symptoms of a sexually transmitted disease, pelvic infection or any inflammation in the pelvic area. No medical claims are warranted or implied by the use of this product.*

♥ Using KRUSH


Preparing to krush

1. Always wash the product before and after each use with antibacterial soap and water. Dry thoroughly. (This product is waterproof.)
2. Get comfortable and relax. A reclined or lying down position is highly recommended.
3. Press  to turn on. LED will light up indicating the product is on. (See diagram)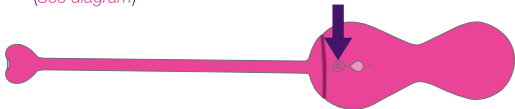

Positioning krush

1. Add lubricant to the product for easy insertion. A water-based lubricant is recommended, such as Lovelife Natural Intimate Lubricant.
2. Fully insert the product into your vagina **keeping the power button facing upward** and the **retrieval cord outside the body**.

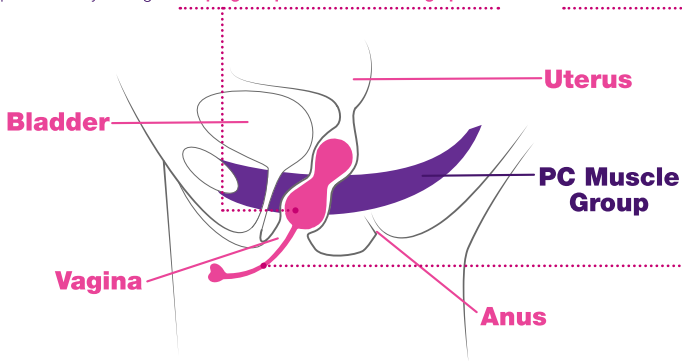
Tip: The retrieval cord should remain outside the body by at least 1 inch to maintain the Bluetooth® connection to the App.

3. The PC muscle is located roughly 1 inch inside the vagina. To ensure that the product remains in position throughout your Kegel exercise, use the positioning anchor provided to keep the product in place. This will ensure that the product can measure your PC muscle activity properly.


4. Place the positioning anchor on the retrieval cord.




5. Slide the position anchor up the vaginal opening making sure it remains outside the body.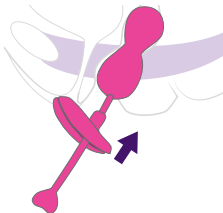

6. Gently tug on retrieval cord so that the larger ball is positioned inside the vagina where your PC muscle is located.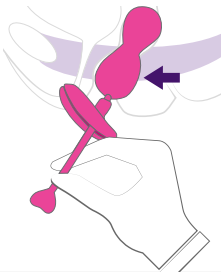

7. Lock the retrieval cord onto the positioning anchor.

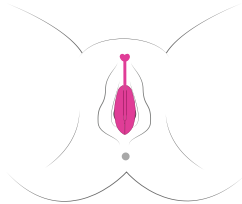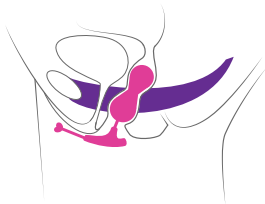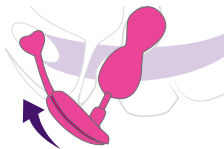
Ready to krush


1. Download tasl app.




2. Pair krush to App simply by following the on-screen instructions.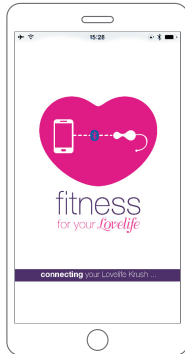

3. Follow quick start guide to get started.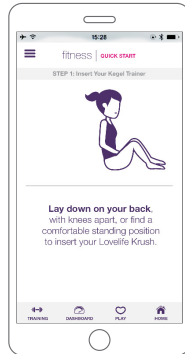


4. For additional information, please refer to the tasl main menu.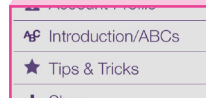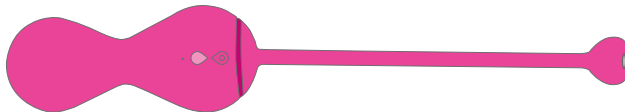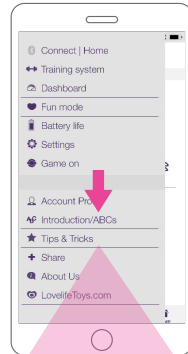

Using krush without the App/smartphone (default mode)

For **Kegel exercise mode** with vibratory feedback only


1. Turn on product.
2. Position product inside the vagina per instructions.
3. Contract and relax pelvic floor muscles to receive vibratory feedback that you are doing your exercises correctly.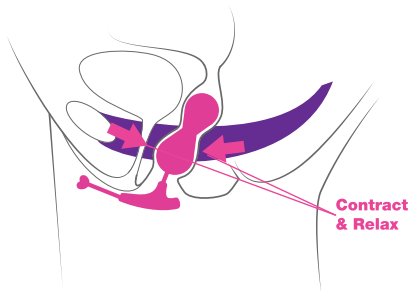

4. To turn off, press & hold  for 3-4 seconds.

For **pleasure** and **constant vibration mode**

1. Turn on product.
2. Press the button once more to switch to constant vibration. To switch back to vibratory feedback mode, simply press the power button once again.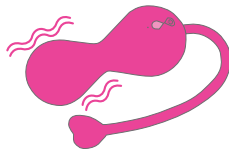

OR


To pair with and control by your phone/tablet, within Bluetooth® range or over wifi, download and launch the OhMiBod Remote App. Additional support can also be found at: www.ohmibod.com/app

3. To turn off, press & hold  for 3-4 seconds.

♥ Removing KRUSH

To remove krush, simply wrap the retrieval cord around your finger and gently tug on the retrieval cord to remove from the vagina.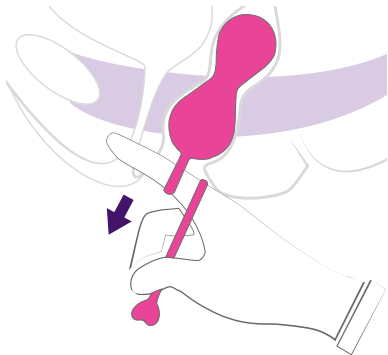

♥ Charging KRUSH

1. Simply plug the USB charging cable into your computer or UL certified USB wall charger (with output voltage 5V) and gently connect the other end of the cable to the magnetic charging socket (See diagram).

2. The LED light will remain on while charging. The LED light will turn off when the product is fully charged.

Tip: Prior to first use, please charge the battery for 4 hours.

NOTICE

Additional charging safeguards:

- Please be sure that the product is completely dry before charging.
- Only use the charging cable provided to charge the product.
- Do not use USB charging hubs to charge the product.
- Only use with UL certified USB wall chargers.

♥ Battery Disposal

- This product contains a rechargeable battery. Please do not dispose of this product with normal household waste.
- Please dispose of this product at the appropriate electrical and electronic equipment recycling/ disposal locations in accordance with government regulations.
- The recycling of this product appropriately will help to protect our environment.

♥ Care & Cleaning

- Clean before and after every use with antibacterial soap and water. Dry thoroughly. This product is waterproof.
- Store in privacy pouch to keep clean and safe until next use. Avoid direct sunlight and keep product away from children.

♥ Important Safeguards

- Do not use on areas that are swollen, inflamed or have skin lacerations.
- Stop using the product if you experience any pain or discomfort.
- If discomfort continues to occur, consult your physician.
- Kegel exercise should never exceed more than 60 minutes in one day.

♥ Specifications

♥ **Materials:**

Body safe silicone

♥ **Size:**

1.3" diameter, 7.1" length
(33mm diameter, 205mm length)

♥ **Weight:**

1.9 oz (54g)

♥ **Battery:**

Lithium-ion 3.7V with 200mAH

♥ **Charging:**

1 hour

♥ **User time:**


Up to 4 hours

♥ **Stand-by time:**

Up to 90 days

♥ **Waterproof:**

IP67 - Submerge up to 1 meter

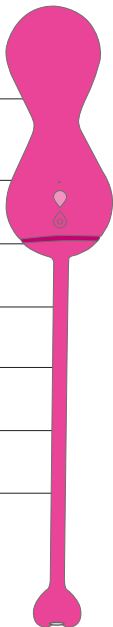 Warranty

OhMiBod warrants this product for a period of one (1) year, from the date of original purchase, against defects due to faulty workmanship or materials. In the event that this product fails during the warranty period, simply email us at customercare@ohmibod.com and provide us with a valid proof of the date of original purchase and we will replace the faulty product at no extra charge.

This warranty does not cover cosmetic deterioration caused by normal wear and tear or damage caused by accident, misuse, neglect or other use not in accordance with this instruction manual. Any attempt to open or take apart this product or its accessories will void the warranty.

 Contact Us

If you have any questions regarding your Lovelife krush product, please check out additional information at www.ohmibod.com. There you will find a downloadable electronic copy of this manual. You can also reach us via email at customercare@ohmibod.com.

Disclaimer: *Users of this OhMiBod product do so at their own risk. Neither OhMiBod nor its retailers assume any responsibility or liability for the use of this product. OhMiBod may make changes to improve this product without notice. OhMiBod may also revise this instruction manual and reserves this right without obligation to notify any person.*

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Copyright© Suki, LLC 2019. All rights reserved.