

Introduction

Thank you for using OTOUCH's products! For better use of this product, please read this manual carefully.

Packing list

- 1 x Penis Pump
- 2 x Seal Rings
- 1 x Charging Cable
- 1 x Instructions

Technical Parameter

Material: Silicone, ABS, GPPS

Air Exhaustion: Electric

Working Voltage: 3.7V

Battery Capacity: 420mAh

Working Noise: <60dB

Duration: >60 minutes (full power state)

Keyboard Function



1. Long press the button for two seconds to turn the power supply on or off.
2. Press to start or stop air exhaustion.
3. The color of the indicator light changes with the intensity of negative vacuum pressure. (Blue is the starting state, and red is the strongest)



- Press the release button to discharge the negative vacuum pressure.

Charging Instructions

Before initial use, make sure to charge the product. Insert the provided USB charging cable into the product's charging port. Use either a computer's USB port or a charger with a USB interface to charge the product. The indicator light will flash during normal charging and remain steady when the battery is fully charged.

Use Directions

1. Apply water-soluble lubricant to where the penis enters or the sealing ring.
2. Insert the non-erect or semi-erect penis into the vacuum assistant through the usage entrance of the sealing ring.
3. Long press the on/off button for two seconds to turn the power on, then press again to turn the pumping power on.
4. Start suction slowly by feeling a gentle pulling sensation.
5. Once the penis is erect, quickly press the button to stop pumping. Press and hold the release button to discharge the pressure.
6. In case of pain or discomfort, press and hold the release button immediately to discharge the pressure and stop using.

Cleaning Care

1. Clean the sealing ring with warm water and antibacterial soap before and after each use.
2. Gently clear the water with a dry towel and keep it air dry.
3. Do not allow water to enter the electric device, as it is not waterproof.
4. Use only water-soluble lubricants, not silicon lubricants.
5. Keep in a clean place away from direct sunlight and out of reach of children.
6. When the inner cylinder is stained and needs to be cleaned, unscrew the electric device to clean the cylinder and the sealing ring.

Health and safety precautions

- The device should not be operated while charging. Please avoid turning it on during the charging process.
- Apply only the minimum vacuum pressure required for an erection; overuse may damage the penis.
- Stop using the vacuum aid if you experience pain.
- Do not use under the influence of alcohol or drugs, which may impair judgment and increase the risk of penile injury.
- Do not use if you have sickle cell disease, a history of prolonged erections, or take large amounts of aspirin or other blood thinners, as these conditions can increase the risk of bruising and hematoma.
- Use may bruise or rupture blood vessels in the penis or scrotum, resulting in petechiae, bleeding, or hematoma formation.
- Use may aggravate pre-existing conditions such as Peyronie's disease (the formation of sclerotic tissue in the penis that causes pain, bending and twisting, usually during erection); Priapism (persistent, painful erection of the penis, usually due to a disease, unrelated to sexual arousal); and urethral stricture (urethral stricture is an area of hardened tissue that narrows the urethra, sometimes making urination difficult).
- Incorrect use of vacuum aids can damage the penis.
- If you have symptoms of erectile dysfunction, that is, an inability to maintain a sufficient erection for intercourse, please consult a doctor before using this device. That way, you can avoid potentially harmful delays in diagnosing the most common causes of this condition, such as diabetes, multiple sclerosis, cirrhosis, chronic renal failure, or alcohol abuse.
- Do not use this product if you experience pain in the penis area.
- Do not use the product if your hand strength is inhibited and you cannot operate normally.

Sketch Map

