

# plusOne / CVS

6770N Menopause Massager

240607

Print Production

3x5" card

## ALL COLORS CMYK

- Uncoated Cardstock
- Bleed
- Double sided
- Tri-fold



CYAN



MAGENTA



YELLOW



BLACK

## NON-PRINTING COLORS



DIELINE



SUBSTRATE

— = Indicates fold

Back

Front

## menopause massager

### cleaning:

Clean thoroughly with mild soap, or plusOne® foaming device cleaner, and warm water, before and after each use.

### storage:

Store in a cool dry place separately from other sexual wellness devices; sexual wellness devices of different materials can damage each other if left in contact. Includes storage bag to hold the device and charging cord.

### quality:

This product is manufactured using body-safe materials that are harmless to the human body; free from phthalates, BPA and latex.

### special instructions:


Do not turn on while charging and/or cleaning. The light will flash while charging, then turn solid once fully charged. When the battery's low, the light will flash rapidly. Use only with 5-volt USB wall adapter (standard mobile phone charger). Use water-based lubricant only. Allow 120 minutes for a full charge.

### warnings:

Discontinue use if there is any discomfort for you or your partner. Do not expose device to water while charging. Do not use while charging. Use only with provided plusOne® charging cable.

# plusOne®

Scan for 2 year warranty registry

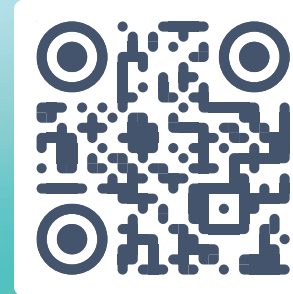

## wellness in your hands

To learn more about ways to use the menopause massager, visit [myplusOne.com](http://myplusOne.com) or follow us on social

[@plusOneclio](#) [@\\_myplusOne](#)

[@\\_myplusOne](#)

# plusOne®


## menopause massager



### instruction manual

Outside

### product overview

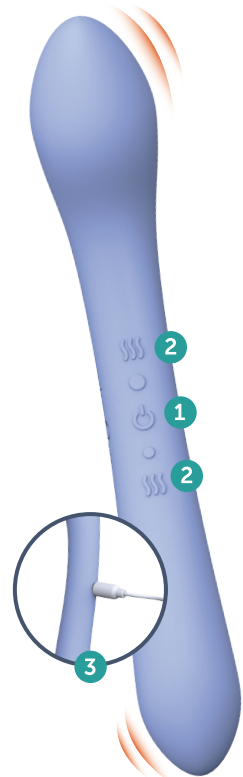

#### 1. power button:

Press and hold the power button for 3 seconds to turn on/off. Press the circle buttons above and below the power button to toggle through the 10 vibration settings on each end.


#### 2. heat buttons:

Press and hold the heat buttons for 3 seconds to turn on/off. The LED light will pulsate to show it is heating up and turn solid when fully warm.

#### 3. charging port:


Attach the magnetic charging head to the charging contact.

### how to charge

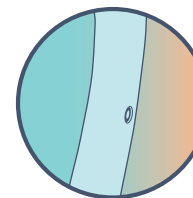


#### step 1

The charging contact is a small metal circle located on the back of the device.

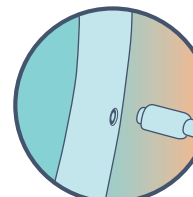

#### step 2

Line up the magnetic charging head with the charging contact.

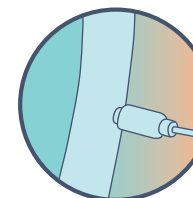


#### step 3

The cable will magnetically attach itself in place. The LED light will pulsate to indicate charging.


The battery shall be disposed in accordance with the local laws and regulations. If possible, please recycle it. Users shall not take out or replace the battery by themselves; doing so may damage the product.

### ways to use

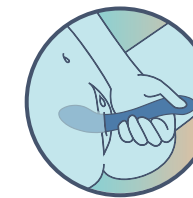


#### vaginal wellness

Use this device as a tool to dilate the vaginal canal and prevent dryness caused by perimenopause & menopause. Be sure to use personal lubricant!

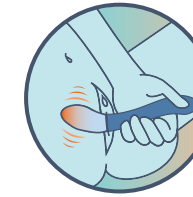

#### get warmed up

Warming technology heats up right above body temperature for a comforting feel that also stimulates blood flow.

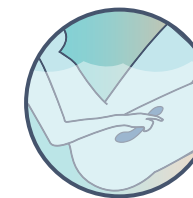

#### create a relaxing oasis

Pick your ideal size and toggle through a variety of vibration settings for total relaxation and satisfaction.

**INCLUDES:** menopause massager, storage bag, charging cable, and instruction manual

Inside