

# plusOne / Walmart/ CVS

6769 Breast & Body Massager

Instruction Manual

231218

Print Production

5x3" Card

All colors are CMYK

- Uncoated Cardstock

- Bleed

- Double sided

- Tri-fold

= Indicates fold

Back

Front

# plusOne®

## breast & body massager

### cleaning:

Clean thoroughly with mild soap and warm water or plusOne® cleaning wipes, before and after each use.

### storage:

Store in a cool dry place separately from other sex toys; sex toys of different materials can damage each other if left in contact. Includes storage bag to hold the device and charging cord.

### quality:

This product is manufactured using body-safe silicone that is harmless to the human body; free from phthalates, BPA and latex.

### special instructions:

Do not turn on while charging and/or cleaning. The light will flash while charging, then turn solid once fully charged. When the battery's low, the light will flash rapidly. Use only with 5-volt USB wall adapter (standard mobile phone charger). Allow 120 minutes for a full charge.

### warnings:


Discontinue use if there is any discomfort for you or your partner. Do not expose device to water while charging.

for complete user manual & more information on ways to use your new breast & body massager:

visit [myplusOne.com](http://myplusOne.com)

or follow us: [@\\_myplusOne](https://www.instagram.com/_myplusOne)

## DON'T MISS YOUR EXCLUSIVE WELCOME OFFER

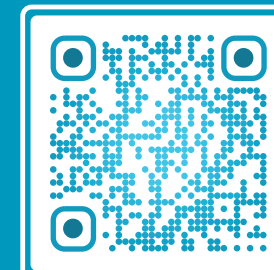

scan to learn more

unlock on [myplusOne.com](http://myplusOne.com) when you sign up via email to receive news and promotions

love your purchase? let us know!



# plusOne®

## breast & body massager



## instruction manual

Outside

## product overview

### 1) power button:

Press and hold the power button for 3 seconds to turn on/off. Press the power button to toggle through the 5 vibration settings.



### 2) charging port:

Attach the magnetic charging head to the charging contact.


### 3) heat button:

When the product is turned on, press and hold the heat button for 3 seconds to turn on/off. The LED light will pulsate to show it is heating up and turn solid when fully warm. Press the heat button to toggle through the 3 heat settings.


The battery shall be disposed in accordance with the local laws and regulations. If possible, please recycle it. Users shall not take out or replace the battery by themselves; doing so may damage the product.

**INCLUDES:** breast & body massager, charging cable, storage bag, and instruction manual


## how to charge

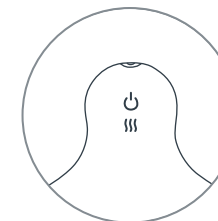

1. The charging contact is a small metal circle located on the top of the device.

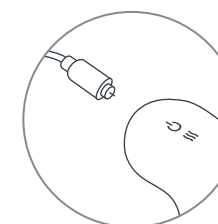


2. Line up the magnetic charging head with the charging contact.

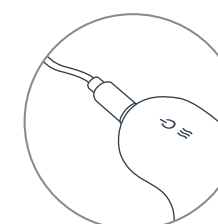

3. The cable will magnetically attach itself in place, and the LED light will pulsate.

## ways to use

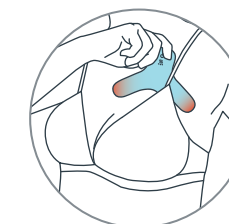

### get warmed up

Start by placing the device inside your bra and use the lowest heat setting to test skin sensitivity. Leave the device inside the bra for comfort prior to nursing or pumping.



### ease breastfeeding & pumping

Moms— this one's for you! Use this massager's broad curved end for larger areas of the breast and body. Press into the breast while moving towards the nipple to help move milk forward and prevent clogging.



### targeted stimulation

Use the targeted tips for digging in and massaging stubborn areas of the breast and body.

For general pain and cramp relief, adjust heat and vibration to desired settings and apply massager's broad curved head or targeted tips to affected areas.

Inside