

# plusOne / CVS

6774N Vaginal Dilator Set

240329

Print Production

3x5" card

## ALL COLORS CMYK

- Uncoated Cardstock


- Bleed

- Double sided

- Tri-fold

 CYAN

 MAGENTA

 YELLOW

 BLACK

## NON-PRINTING COLORS

 DIELINE

 SUBSTRATE

- - - - -  
= Indicates fold

Back

Front

### vaginal dilator set

#### cleaning:

Clean thoroughly with mild soap and warm water, or plusOne® cleaning wipes, before and after each use.

#### storage:

Store in a cool dry place separately from other sex toys; sex toys of different materials can damage each other if left in contact. Includes a storage bag that holds all four layers, the handle, and charging cord.

#### quality:

This product is manufactured using body-safe materials that are harmless to the human body; free from phthalates, BPA and latex.

#### special instructions:

Do not turn on while charging and/or cleaning. The light will flash while charging, then turn solid once fully charged. When the battery's low, the light will flash rapidly. Use only with 5-volt USB wall adapter (standard mobile phone charger). Allow 120 minutes for a full charge.

#### warnings:

Discontinue use if there is any discomfort for you or your partner. Do not expose device to water while charging. Do not use while charging. Use only with provided plusOne® charging cable. Do not use if you have fragile or sensitive vaginal walls, or if you have an active disease or infection. Stop use immediately if you experience an increase in burning or pain. Please contact your healthcare professional for any questions about proper dilation usage or concerns when using this product.

plusOne®

Scan for 2 year warranty registry




wellness  
in your hands

For the complete user manual & to learn more about ways to use the dilator set, visit [myplusOne.com](http://myplusOne.com) or follow us on social



plusOne®  
vaginal dilator set

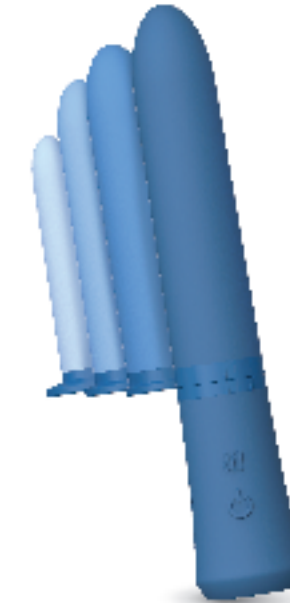

instruction manual

Outside

### product overview

#### 1. power button

Press and hold the power button for 3 seconds to turn on/off. Press the power button to toggle through 10 vibration settings.

#### 2. heat button

When the product is turned on, press and hold the heat button for 3 seconds to turn the heat on/off. The LED light will pulse to show it's heating up and turn solid when fully warm.



#### 3. charging port


Attach the magnetic charging head with the charging contact.

#### 4. handle


To attach and detach the different layers, twist and ensure the markers are properly aligned.



### how to charge

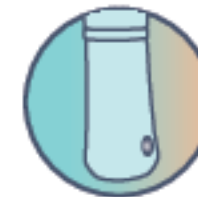

**step 1**  
Locate the charging contact, a small metal circle, located on the back of the handle.

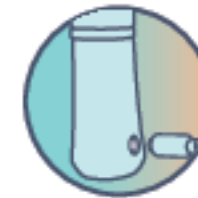

**step 2**  
Line up the magnetic charging head with the charging contact.



**step 3**  
Once the cable magnetically attaches itself in place, the LED light will pulse.

The battery shall be disposed in accordance with the local laws and regulations. If possible, please recycle it. Users shall not take out or replace the battery by themselves; doing so may damage the product.

### ways to use



**pick your insert**  
Four sizes gradually increase in diameter to comfortably expand your vaginal canal. We recommend starting with the smallest size and level up when comfortable. For the best results, use the device 3-5 days a week for 5-15 minutes.



**get warmed up**  
Warming technology heats up right above body temperature for a comforting feel that also stimulates blood flow.



**unwind and relax**  
Pick your ideal size and toggle through a variety of vibration settings for total relaxation and satisfaction.

INCLUDES: vaginal dilator set, storage bag, charging cable, and instruction manual

Inside